The SIMPULSE™ PLUS System

Includes:

• SIMPULSE™ PLUS Suction/Irrigator

• Suction Tubing

• SIMPULSE™ PLUS Tip

• Dual Spike Adapter

ENGLISH

Instructions for Use

R_x Only

Do Not Re-Use

STERILE

EO

Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

Caution: Read all instructions prior to use.

Indications

The SIMPULSE™ PLUS Suction/Irrigator is designed to provide controlled powered irrigation during open surgical procedures (e.g., orthopaedic procedures) or for cleansing of burns/wounds. The pulsatile action of the pump helps to remove blood, tissue debris and foreign matter from the operative/wound site. When connected to a suction source the device can be used to aspirate fluids from the operative/wound site.


Principle of Operation

The SIMPULSE™ PLUS Pump diaphragm is activated by pressure from a compressed nitrogen/air source. Flow is initiated by adjusting the pressure regulator at the compressed nitrogen/air source between 20 to 80 psi until the desired flow rate and pulsing force is achieved.

Do not exceed 80 psi maximum source pressure.

Warnings

1. This device has been designed for single use only. Reuse, reprocessing, resterilization or repackaging may compromise the structural integrity and/or essential material and design characteristics that are critical to the overall performance of the device and may lead to device failure which may result in injury to the patient. Reuse, reprocessing, resterilization or repackaging may also create a risk of contamination of the device and/or cause patient infection or cross infection, including, but not limited to, the transmission of infectious diseases from one patient to another. Contamination of the device may lead to injury, illness or death of the patient or end user.
2. Do not exceed 80 psi maximum source pressure.
3. The exhaust line must be open to atmosphere. Do not connect, clamp or obstruct the exhaust line.
4. If there is leakage of irrigation fluid from the exhaust line either while priming, during set-up or during use, discard and replace the system.
5. Do not connect nitrogen/air connector to CO₂ gas because the gas will be vented into the room atmosphere.
6.  After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws (U.S.A.) and regulations.

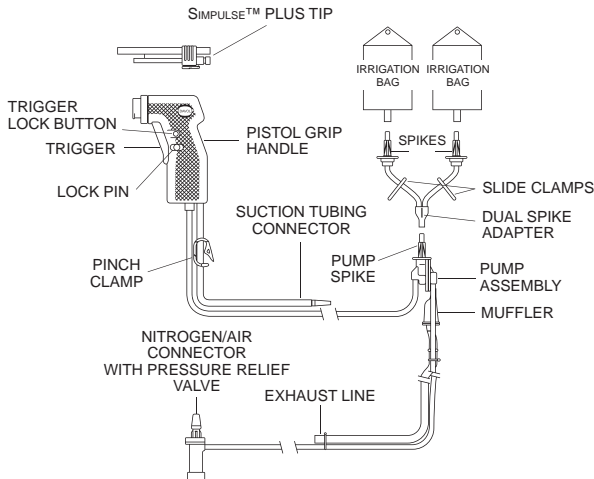SIMPULSE™ PLUS Tip configuration will vary according to product code (REF) number.

Set-up (Additional Equipment Required)

To set up the system you will need the following:

- Compressed nitrogen/air source
- Schrader style connector
- Irrigation Bags

Step 1: Open the Package

1. The circulating nurse opens the package and delivers the contents onto the sterile field.

Step 2: Connect the Irrigation Fluid

1. The scrub nurse:
 - a) Connects the dual spike adapter tubing to the SIMPULSE™ PLUS pump spike and passes to the circulating nurse.
 - b) Removes the lock pin to release the trigger.
 - c) Inserts the SIMPULSE™ PLUS tip. Attaches suction tubing onto suction tubing connector, if desired.

NOTE: For Product Code REF 0057700. Remove tip protector before using tip.

To retract splash shield: Grasp from top, push and rotate downward past retaining ring.

To reposition splash shield: Grasp from bottom, pull and rotate upward past retaining ring.

2. The circulating nurse spikes the irrigation bags with the SIMPULSE™ PLUS dual spikes. (McGAW™ and ABBOTT™ bottles will need their vented spike adapters.)
3. The scrub nurse primes the unit by squeezing the trigger until irrigant exits the tip.
4. The circulating nurse may squeeze the irrigation bag(s) to facilitate priming.
5. The circulating nurse attaches the suction tubing to suction source.

Step 3: Connect to Compressed Nitrogen/Air Source

Warning: Do not connect nitrogen/air connector to CO₂ gas because the gas will be vented into the room atmosphere.

1. The circulating nurse reduces the gas source pressure to zero.
2. Insert the nitrogen/air connector into the Schrader style check unit until a click is heard.
3. Set the source pressure to 80 psi for maximum flow and pulsing force. The pressure may be adjusted lower as desired.

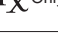
Step 4: Use

1. To irrigate, squeeze the handpiece trigger. A staccato sound will be heard. If a constant high-pitched sound occurs, recheck the source pressure setting. Pressure in excess of 80 psi may activate the pressure relief valve. If this occurs, disconnect the nitrogen/air connector and repeat **Step 3: Connect to Compressed Nitrogen/Air Source.**
2. The trigger lock button may be used to lock the trigger at maximum flow for uninterrupted irrigation. While squeezing the trigger, push the trigger lock button into the handle until it engages. To release the lock, squeeze the trigger.
3. To change tips, grasp tip and pull out straight from handle. Insert desired tip.
4. Close the white pinch clamp on the suction line to shut off suction as desired.

After Use

Reduce pressure regulator to 0 psi at the compressed nitrogen/air source. Detach unit from compressed nitrogen/air source. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws (U.S.A.) and regulations.

Bard, Davol and Simpulse are trademarks and/or registered trademarks of C. R. Bard, Inc. McGaw and Abbott are trademarks of their respective owners. Copyright © 2004, 2008, 2010, 2011, 2015 C. R. Bard, Inc. All Rights Reserved.

	SIMPULSE™ PLUS System		Femoral Separable Tip
	SIMPULSE™ PLUS Suction/Irrigator		Retractable Splash Shield Tip
	Simpulse™ PLUS Tip(s)		Retractable Small Splash Shield Wound Tip
	Suction Tubing		Do not use if package is damaged
	Dual Spike Adapter		Contents
	Soft Splash Shield Tip		Contains phthalates
	Showerhead Tip		Contains or presence of natural rubber latex
	Femoral Tip		U.S. Federal law restricts this device to sale by or on the order of a physician.

 **Manufacturer:**
Davol Inc.
Subsidiary of C. R. Bard, Inc.
100 Crossings Boulevard
Warwick, RI 02886, USA
1-401-825-8300 • 1-800-556-6275
Assembled in Mexico

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Bard, Davol et Simpulse sont des marques commerciales et / ou des marques déposées de C. R. Bard, Inc. McGaw et Abbott sont des marques commerciales de leurs propriétaires respectifs. Copyright © 2004, 2008, 2010, 2011, 2015 C. R. Bard, Inc. Tous droits réservés.
Filiale de C. R. Bard, Inc.