StatLock® Foley Stabilization Device
Read carefully before use.

Safety and Efficacy Considerations:

 Single use. **Rx only** **CAUTION:** Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

STERILE/EO Sterilized using ethylene oxide.

Sterile unless package is opened or damaged, except for any individually packaged components within the pouch which are not labeled as sterile. These components are not terminally sterilized.

Do not alter the **StatLock®** device or components. Procedure must be performed by trained personnel with knowledge of anatomical landmarks, safe technique and potential complications.

Contents: Package includes the **StatLock®** device stabilization system and skin preparation pad.

Indications for Use:


The **StatLock®** device is a stabilization device for compatible catheters.

Contraindications:

Known tape or adhesive allergies.

Warnings and Precautions:

1. Do not use the **StatLock®** device where loss of adherence could occur, such as with a confused patient, diaphoretic or non-adherent skin, or when the access device is not monitored daily.
2. Observe universal blood and body fluid precautions and infection control procedures, during application and removal of the **StatLock®** device.
3. Minimize catheter manipulation during application and removal of the **StatLock®** device.
4. **Daily Maintenance:**
 - a. The **StatLock®** device should be assessed daily and changed when clinically indicated, at least every seven days.
 - b. If pad becomes soiled, wash with soap/water, saline or hydrogen peroxide. Do not use alcohol or prepackaged bathing systems, which could lead to early lifting.
 - c. If showering/bathing, cover with plastic wrap or waterproof dressing.
 - d. Conduct skin assessment prior to application and repeat daily per facility protocol.
 - e. Use clinical judgment on the removal of the **StatLock®** Stabilization device if the patient experiences any fluid shifts that may interfere with skin integrity.

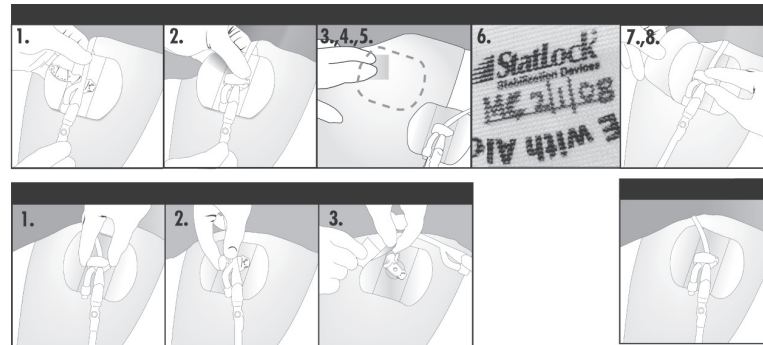Application Technique

Prep

1. Place Foley catheter into retainer. Directional arrow should point towards catheter tip, and balloon inflation arm should be next to the clamp hinge.
2. Close lid, being careful to avoid pinching the catheter.
3. Identify securement site by laying the device retainer on the front of the thigh, leaving 1 inch of catheter slack between insertion site and the **StatLock®** device retainer.
4. After placing the **StatLock®** Stabilization device off to the side, cleanse and degrease the securement site with alcohol per hospital policy. Let skin dry.
5. Apply skin protectant, in direction of hair growth, to area larger than securement site. Allow to dry completely (10-15 seconds).
6. Using permanent marker, write initials and date of application on the **StatLock®** device anchor pad.

NOTE: Always secure catheter into the **StatLock®** device retainer before applying adhesive pad on skin.

Place and Peel

7. Align the **StatLock®** Stabilization Device over securement site leaving 1 inch of catheter slack. Make sure leg is fully extended.
8. While holding the retainer to keep the pad in place, peel away paper backing, one side at a time and place tension-free on skin.

Removal Technique

Disengage

1. Open retainer by pressing release button with thumb, then lift to open.
2. Remove Foley catheter from the **StatLock®** device.

Dissolve

3. Wipe the edge of the pad using at least 5-6 alcohol pads until a corner lifts. Then continue to stroke undersurface of pad with alcohol to dissolve adhesive pad away from skin.

Do not pull or force pad to remove.

*The **StatLock®** Stabilization device should be monitored daily and replaced when clinically indicated, at least every 7 days. The catheter insertion site should be treated per local policy recommendations. The **StatLock®** device is contraindicated on patients with known tape and adhesive allergies. Alcohol and acetone may weaken the adhesive bond between the **StatLock®** device pad and the skin. Care should be taken when using these solutions while performing catheter site care. Please consult product inserts and labels for any indications, contraindications, hazards, warnings, cautions and directions for use. For additional information on the **StatLock®** Foley stabilization device, please call your local Bard Representative.*

Manufacturer:
C. R. Bard, Inc.
Covington, GA 30014 USA
800-526-4455
www.bardmedical.com

Assembled in Mexico

© Copyright 2006-2017 C. R. Bard, Inc.
Bard and StatLock are trademarks and/or registered trademarks of C. R. Bard, Inc.
Protected by domestic and foreign patents and patents pending.

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