



**ENGLISH**

## Catheter Stabilization Device

**NON-STERILE. READY TO USE.**

**[REF]** Product code, **[LOT]** lot number and **[X]** expiration date information—see packaging.

Remove oil and moisturizer from targeted skin area. Apply skin prep to the targeted **StatLock**® device area for skin protection and enhanced pad adherence.

**ALLOW TO DRY COMPLETELY.**

### **Safety and Efficacy Considerations:**

⊗ Single use only. Rx Only.

**LATEX FREE**

### **Contraindications:**

Known tape or adhesive allergies.

## **Warnings and Precautions:**

Avoid contacting the **StatLock**® device with alcohol or acetone, both can weaken bonding of components and the **StatLock**® device pad adherence.

**BARD**

C. R. Bard, Inc.  
Covington, GA 30014  
1-800-526-4455  
[www.bardmedical.com](http://www.bardmedical.com)  
[www.statlock.com](http://www.statlock.com)


Assembled in Mexico

Bard and StatLock are trademarks and/or registered trademarks of C. R. Bard, Inc. or an affiliate. Protected by domestic and foreign patents and patents pending.

PK2401881 02/09

## Dispositivo de fijación de catéteres

NO ESTÉRIL. LISTO PARA SU USO.

**REF** Para información sobre el código del producto, **LOT** el número de lote y  la fecha de caducidad, véase el envase.

Eliminar el aceite y las cremas hidratantes de la zona de la piel donde se va a aplicar el dispositivo **STATLOCK**<sup>®</sup>. Aplicar el preparado para la piel al área de aplicación del dispositivo **STATLOCK**<sup>®</sup> a fin de proteger la piel y lograr una mayor adherencia del parche. **DEJAR SECAR COMPLETAMENTE.**

## Consideraciones acerca de la seguridad y la eficacia:

 Para un solo uso

**NO CONTIENE LÁTEX**

## Contraindicaciones:

Alergias conocidas a los adhesivos o esparadrapos.

## Advertencias y precauciones:

Evitar el contacto del dispositivo **STATLOCK**<sup>®</sup> con alcohol o acetona, ya que pueden debilitar la unión de los componentes y la adherencia del parche del dispositivo **STATLOCK**<sup>®</sup>.

**BARD**

C. R. Bard, Inc.  
Covington, GA 30014  
1-800-526-4455  
[www.bardmedical.com](http://www.bardmedical.com)  
[www.statlock.com](http://www.statlock.com)

Fabricado en México

**BARD** y **STATLOCK** son marcas comerciales y/o comerciales registradas de C. R. Bard, Inc. o de una filial.

Protegido por patentes nacionales y extranjeras, y patentes pendientes.