

BARD® Urological Catheter

X-Ray Opaque Rubber

For urological use only

Warnings:


- Do not use ointments or lubricants having petrolatum base. They will damage the latex.
- After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.


Cautions:

- Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions
- Federal (U.S.A.) laws restricts this device to sale by or on the order of a physician
- Sterile unless package is opened or damaged. Do not use if package is opened or damaged

Visually inspect the product for any imperfections or surface deteriorations prior to use.

Storage: Store catheters at room temperature away from direct exposure to light, preferably in the original box.

 Do not reuse.

 Do not resterilize.

BARD

 Manufacturer:
C. R. Bard, Inc.
Covington, GA 30014
1-800-526-4455
www.bardmedical.com

