BARD

UltraFlex™

SELF-ADHERING MALE EXTERNAL CATHETER
Spirit® Style 1
HYDROCOLLOID ADHESIVE SHEATH

DESCRIPTION/INDICATION
The self-adhering male external catheter is designed for the management of male urinary incontinence.

CONTRAINDICATION
Do not use on irritated or compromised skin.

PRECAUTION
Do not use if allergic reaction occurs. For good hygiene, change catheter daily. Use of a single device for longer periods than 24 hours may increase the risk of complications.

DIRECTIONS: TO APPLY
1) Wash penis with mild soap and warm water. Dry thoroughly.

- Trim pubic hair if necessary.
- Unroll self-adhering catheter over penis.
- Gently squeeze the catheter to properly seal adhesive to the skin.

IMPORTANT: Wear time may be significantly reduced if adhesive is not properly sealed to the skin.

5) Connect to drainage device.

DIRECTIONS: TO REMOVE
Gently roll catheter off the penis.

Note: If necessary, apply a warm wet compress (such as a wet washcloth) around the catheter to help loosen the adhesive.

REF

LOT

Catalog Number

Lot Number

EC REP

Manufacturer

Authorized Representative

Units

Non-Sterile

Single Use

Use By

Not made with natural rubber latex

CAUTION: consult accompanying documents

EC REP

Bard Limited, Forest House
Crawley, West Sussex UK
RH11 9BP
+44 1293 527 888

Bard, Spirit and UltraFlex are trademarks and/or registered trademarks of C. R. Bard, Inc. ©2016 C. R. Bard, Inc. All Rights Reserved. PK2450030 01/2017

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