BARD

Pop-On™

SELF-ADHERING MALE EXTERNAL CATHETER

Wideband™

SELF-ADHERING MALE EXTERNAL CATHETER

Spirit®


Style 2 / Style 3
HYDROCOLLOID ADHESIVE SHEATH

DESCRIPTION/INDICATION
The self-adhering male external catheter is designed for the management of male urinary incontinence.

CONTRAINDICATION
Do not use on irritated or compromised skin.

PRECAUTION
Do not use if allergic reaction occurs. For good hygiene, change catheter daily. Use of a single device for longer periods than 24 hours may increase the risk of complications.

- DIRECTIONS: TO APPLY**
- 1) Wash penis with mild soap and warm water. Dry thoroughly.
 - 2) Trim pubic hair if necessary.
 - 3) Open package at perforation.
 - 4) To remove plastic insert, squeeze catheter at the top of the white cone and pull to release.

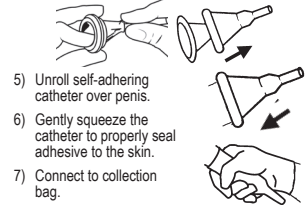IMPORTANT:
Wear time may be significantly reduced if adhesive is not properly sealed to the skin.

DIRECTIONS: TO REMOVE
Gently roll catheter off the penis.
Note: If necessary, apply a warm wet compress (such as a wet washcloth) around the catheter to help loosen the adhesive.

REF Catalog Number
LOT Lot Number

EC REP
Manufacturer Authorized Representative Units

NON STERILE Non-Sterile
Single Use Single Use
Use By Use By
NRL Not made with natural rubber latex

Manufacturer:
C. R. Bard, Inc.
Covington, GA 30014 USA
(800) 526-4455
www.bardcare.com

CAUTION: consult accompanying documents
CE

Bard, Pop-On, Spirit and Wideband are trademarks and/or registered trademarks of C. R. Bard, Inc. ©2016 C. R. Bard, Inc. All Rights Reserved. PK2450031 01/2017

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