# X-STREAM™ Laparoscopic Irrigation Tubing Set

For use with

X-STREAM™ Laparoscopic Irrigation Controller (REF 5551000, REF 9551000) and optional Dual Spike Adapter (REF 0026110)

Reorder Number **5552000** - X-STREAM™ Laparoscopic Irrigation Tubing Set with SMOKEVAC™ Trumpet Valve, 5mm x 33cm Probe Tip with Holes and Cojoined Suction/Irrigation Tubing

Reorder Number **5552001** - X-STREAM™ Laparoscopic Irrigation Tubing Set with SMOKEVAC™ Trumpet Valve and Cojoined Suction/Irrigation Tubing, without Probe Tip

Reorder Number **5552002** - X-STREAM™ Laparoscopic Irrigation Tubing Set with NEZHAT-DORSEY™ Trumpet Valve, 5mm x 33cm Probe Tip with Holes and Cojoined Suction/Irrigation Tubing

Reorder Number **5552003** - X-STREAM™ Laparoscopic Irrigation Tubing Set with NEZHAT-DORSEY™ Trumpet Valve and Cojoined Suction/Irrigation Tubing, without Probe Tip

## Instructions for Use

Indicated for: LAPAROSCOPY

CAUTION: Read all instructions prior to use.

X-STREAM™ Irrigation System

Do Not Re-use

Rx only

STERILE EO

### Indications:

The X-STREAM™ Irrigation System is designed to provide controlled irrigation to the operative site during laparoscopic procedures. The system helps flush blood and tissue debris from the operative site during laparoscopy to aid visualization. It may also be useful for resection of filmy adhesions (hydrodissection), hydrodis-solution of blood clots and peritoneal lavage.

When fitted with an electro-surgical attachment or insert, the device may be used for cutting, dissection and coagulation of tissue using monopolar or bipolar energy—depending on the electro-surgical accessory utilized. When fitted with the proper adapter and probe tip, the device may be used to introduce a laser fiber or laparoscopic surgical instrument into the peritoneal cavity during a laparoscopic procedure.

### Contraindications:

Use of this device for hysteroscopy or for cavity distention is contraindicated.

Use of this device for intra-abdominal irrigation is contraindicated whenever laparoscopy is contraindicated. See operator's manual of your laparoscope for absolute and relative contraindications. Use of monopolar electro-surgical instruments and inserts with an uninsulated metallic probe tip is contraindicated.

NEZHAT-DORSEY™ and NEZHAT-DORSEY™ SMOKEVAC™ Trumpet Valves:


### Intended Use:

The NEZHAT-DORSEY™, or SMOKEVAC™ Trumpet Valve Irrigation and Suction Tubing Set, when fitted to a DAVOL™ Suction/Irrigation Probe Tip, is intended for the delivery of solution/irrigant to, and aspiration of fluids/smoke from, the operative site during laparoscopic surgical procedures.

Additional specialty probe tips are available from DAVOL™, including insulated and nonconductive tips for use with electro-surgery.

Suction/irrigation probe tips, when included, have a non-reflective surface to help avoid laser beam reflection. With proper adapter, a laser fiber and laser fiber sheath can be passed through the probe handle and tip. Pathology and/or surgeon's choice will dictate preference of suction/irrigation probe tip configuration.

Note: The NEZHAT-DORSEY™ Trumpet Valves offer variable flow control. For example, depressing the irrigation button on the handpiece halfway delivers approximately one-half of the available irrigation pressure/flow.

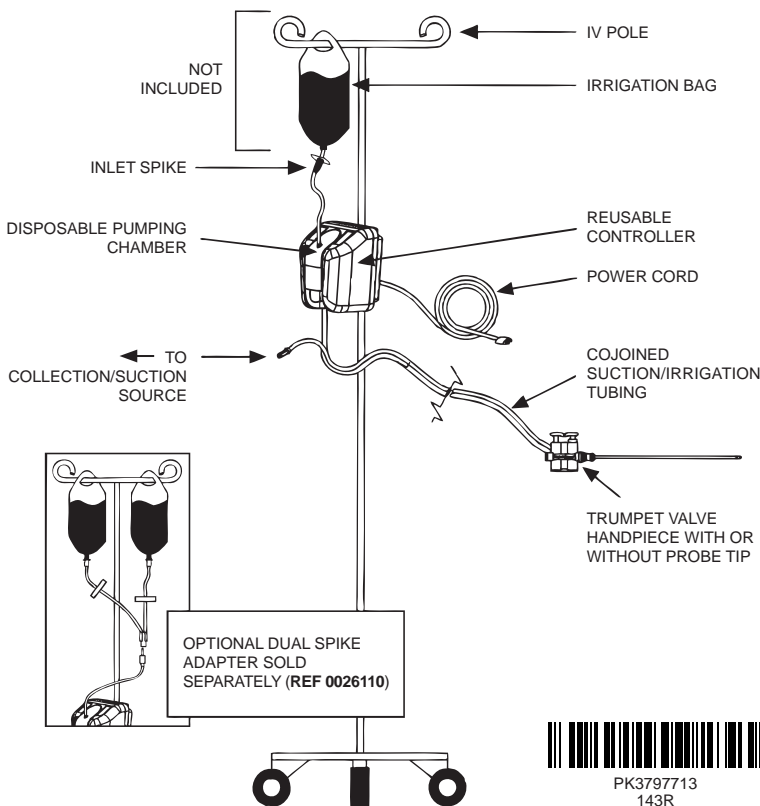PK3797713  
143R

## Principles of Operation

The X-STREAM™ Tubing Sets are intended for use with the electrically powered X-STREAM™ Controller. Irrigation flow rate is determined by one of the two settings on the controller. The X-STREAM™ Controller automatically turns on when the tubing set is placed securely in the pumping chamber receptacle of the controller. The controller will default to a setting that provides an irrigation flow of approximately 3 liters/minute.

The X-STREAM™ Controller is only activated when the irrigation button on the trumpet valve is depressed. The irrigation button on the trumpet valve may be utilized to control the actual amount of irrigation impact force delivered to the surgical site. For example, depressing the irrigation button halfway, will deliver approximately one half of the irrigation impact force offered with a specific pump setting.

## Warnings

- This device has been designed for single use only. Reuse, reprocessing, resterilization or repackaging may compromise the structural integrity and/or essential material and design characteristics that are critical to the overall performance of the device and may lead to device failure which may result in injury to the patient.**  
Reuse, reprocessing, resterilization or repackaging may also create a risk of contamination of the device and/or cause patient infection or cross infection, including, but not limited to, the transmission of infectious diseases from one patient to another. Contamination of the device may lead to injury, illness or death of the patient or end user.
- As with any surgical procedure where extensive irrigation is utilized, intravasation levels should be closely monitored by the attending physician.
- The pressure accuracy of the X-STREAM™ System depends on proper setup. It is important that the bottom of the X-STREAM™ Controller be positioned at least 12 inches above the highest height of the patient to provide sufficient pressure for optimal performance of the system.
- The X-STREAM™ Irrigation System is capable of generating high irrigation impact forces and flow rates. Use the minimum pressure necessary at all times to accomplish the procedure.
- The X-STREAM™ Irrigation System will generate even higher irrigation impact forces (compared to impact forces generated via a 5mm standard probe tip) when used with the DAVOL™ 10mm ENDO-POOL™, 10mm Hydro-Dissection Channel Probe or the 5mm Micro Probe tips.
- Electrosurgery instruments or electrode inserts must not be used with uninsulated metallic probe tips. Electrosurgery instruments and inserts should be used only with available DAVOL™ non-conductive or insulated probe tips.
- When using electrosurgical attachments or inserts, follow all instructions, warnings, precautions, and contraindications provided with these devices.
- The tubing set will provide gravity flow if the pumping chamber is not properly loaded into the controller's pumping chamber receptacle.
- Do not modify this equipment without authorization of DAVOL™, as it may compromise the safety and effectiveness of the device.
- If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe and effective use of the equipment.
- DAVOL™ must be contacted for instructions regarding the correct replacement of interchangeable or detachable parts.

## Precautions

- The controller automatically turns "ON" when the tubing set is loaded into the controller. The logo and PRIME LED's will light to indicate that the controller is on. There is no "ON/OFF" switch.
- If package is damaged or open, do not use product.
- Do not remove the square, metal contact label located at the top back of the pumping chamber.
- The bottom of the controller assembly should be at least 12 inches above the highest height of the patient at any time during surgery.
- Use of Irrigation bottles is NOT recommended because performance is diminished by air being pulled into the controller before it can be vented into the bottle. However, if irrigation bottles are used, simultaneously using two (2) bottles with a "Y" connector (optional Dual Spike Adapter REF 0026110) will alleviate the air entry issue. For optimum performance, use of irrigation bags is recommended.
-  After use, the tubing set, trumpet valve and probe tip are a potential biohazard. Handle and dispose of as required by hospital policy and applicable laws.
- Please refer to the X-STREAM™ Controller Operator's Manual for additional information.
- Adhesions, anatomical anomalies or other conditions may prevent clear visualization of internal anatomy. Do not perform laparoscopic procedures if the internal anatomy can not be positively identified.
- Laparoscopic procedures should be performed only by persons having adequate training and familiarity with laparoscopic techniques. Consult the medical literature relative to techniques, complications and hazards prior to performance of any laparoscopic procedure.
- This product is for single use and is provided sterile. DO NOT RESTERILIZE.
- Follow all of the recommended electrosurgical and laser manufacturers' instructions and warnings while operating in these modes.
- Gravity prime and perform all priming with the probe tip outside the patient.
- Irrigation fluid must have a specific gravity of approximately 1 (e.g. saline).
- Grounding reliability can only be achieved when power supply is connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".
- X-STREAM™ should be positioned so that it is not difficult to disconnect device from the power source, in order to facilitate access.
- The X-STREAM™ controller should not be placed adjacent to or stacked with other equipment unless nominal operation of the X-STREAM™ has been observed as it may cause a malfunction of the device.
- Use special precautions regarding the use of the X-STREAM™. X-STREAM™ is to be used in EMC environments that are suitable for Group 1, Class A medical equipment. See EMC information provided in X-STREAM™ Controller Operator's Manual.

## WHEN PASSING LASER FIBERS THROUGH THE TRUMPET VALVE:

- CO<sub>2</sub> laser energy systems are not compatible for use through the trumpet valve and probe tip.
- A thorough understanding of the principles and techniques of laser surgical procedures is necessary to avoid injury to the patient, operator and operating room personnel.
- Pathology and the surgeon's choice will dictate laser tip configuration.
- Proper eyewear is mandatory before activating the laser.
- Use of this device with laser equipment is indicated only in those procedures for which the selected laser fiber and laser system being used have been approved by the FDA.
- To avoid damage to the laser fiber, do not expose tip until suction/irrigation probe tip has passed through the laparoscopic trocar.

## Tubing Set Instructions For Use

**Important:** Thoroughly review all Instructions for Use for the X-STREAM™ Laparoscopic Irrigation Tubing Set and the X-STREAM™ Laparoscopic Irrigation Controller before using system.

## X-STREAM™ Controller Setup

- With the bottom of the X-STREAM™ Controller at least 12 inches above the highest height of the patient at any time during surgery, mount the controller on the IV pole. To mount, loosen pole clamp knob by turning it. Position the pole clamp around the IV pole and tighten the knob to secure the controller to the pole.
- Ensure the IEC power cord is plugged into the IEC power cord receptacle on the back of the controller. Also ensure that the power cord is connected to an operable Hospital Grade AC outlet.
- Hang irrigation bag(s) on the IV pole. Because of the high flow rates generated by the X-STREAM™ Irrigation System, 2000cc or larger IV bag sizes are recommended to minimize bag changes.
- If spiking two bags is desirable, use the optional Dual Spike Adapter (REF 0026110).  
**Note:** Changing the setting is possible both with and without the motor running.  
**Note:** The flow rate setting cannot be changed when the controller is in the "PRIME" mode.  
**Precaution:** Do not remove the square, metal contact label located at the top back of the pumping chamber.

## Opening The Tubing Set Package

- The circulating nurse opens the package and delivers the tubing set onto the sterile field.
- Remove protective sleeve from disposable bundle.
- The scrub nurse retains the trumpet valve handpiece and checks and tightens the knurled Quick Disconnect Adapter at the front of the Trumpet Valve. The scrub nurse removes the protective sleeve from the included disposable tip (REF 5552000, REF 5552002) and attaches this or another compatible probe tip to the trumpet valve by pushing the probe tip over the Quick Disconnect Adapter until fully seated (on some tips a double click can be felt and/or heard).
- Additional specialty tips are available from DAVOL™, including insulated and nonconductive tips for use with electrosurgery inserts. Probe tips are quickly and easily exchanged at any time. Always inspect probe tips for damage. Do not use if damage is evident.

- The scrub nurse then hands the pumping chamber (X-STREAM™ clear cartridge) to the circulating nurse.
- The circulating nurse peels away the suction tubing from the irrigation tubing and connects the suction tubing to the suction source.

### Loading The Tubing Set

**Note:** Read all directions for the X-STREAM™ Laparoscopic Irrigation Controller before beginning the loading procedure.

- Slide the pumping chamber downward into the controller until it snaps into place flush with the top and face of the controller.  
**Precaution:** The controller automatically turns "ON" when the tubing set is loaded into the controller. The logo and PRIME LED's will light to indicate that the controller is on. There is no "ON/OFF" switch.
- Remove spike cap and spike the irrigation bag. Leave sufficient slack in line to facilitate bag placement/removal. If spiking two bags is desirable, use the optional Dual Spike Adapter (REF 0026110).

### Priming the System

**Note:** The system includes audible and visual indicators to let the user know priming is required. The "PRIME" LED light will flash and the audible indicator will sound when the system detects the presence of the pumping chamber upon initial setup.

**Precaution:** Gravity prime and perform all priming with probe tip outside of patient.

- To prime, depress irrigation button (colored button) until all air is out of the system and only irrigant exits the probe tip.
- Release the irrigation button.
- If the pumping chamber is not primed properly, the "PRIME" text will continue to flash and the audio indicator will continue to sound after the irrigation button is released.  
**Note:** Removal and reloading of the pumping chamber will return the controller to the "PRIME" mode. The DAVOL™ logo LED will illuminate to indicate that the X-STREAM™ Controller is "ON". The "PRIME" text will flash and audio indicator will sound to indicate that the tubing set needs to be primed.
- When ready to irrigate, depress the irrigation button (colored button) on the trumpet valve.  
**Note:** The motor will run when the irrigation button is depressed and will stop after it is released. A safety feature turns the pump off after 15 seconds of continuous use. Release the irrigation button and then re-depress it to activate the pump.
- To suction, depress the unmarked or white button on the Trumpet Valve

### Activating SmokEvac™ (REF 5552000, REF 5552001)

- To activate SmokEvac™ rotate SmokEvac™ mechanism counterclockwise to achieve desired rate of smoke/plume removal. SmokEvac™ feature will remain activated even after suction valve is depressed and released.  
**Note:** An immediate medium-to-fast setting can be achieved by rotating the SmokEvac™ mechanism counterclockwise until it stops and then fully depressing the suction button.
- To deactivate SmokEvac™ rotate mechanism clockwise until a complete stop is reached.

### Changing the Flow Rate Setting

- Once primed, the system automatically defaults to the low setting.
- To increase flow rate, press the button labeled "HI".
- The system can be toggled between the low and high settings throughout the procedure by the circulator.
- For either setting, the surgeon can further control the flow at the handpiece.

### Changing Irrigation Bags

**Note:** The system includes audible and visual indicators to let the user know priming is required. The "PRIME" and "HI" LED lights will blink and the audio indicator will sound when the bag runs out of fluid.

**Precaution:** Perform all priming with the probe tip outside the patient. If the bag(s) runs out of irrigant, the motor will stop running. When air enters the system, the controller returns to the "PRIME" state, the "PRIME" and "HI" text will blink, and the audio indicator will sound.

- Hang a new bag(s) and spike.
- Depress the irrigation button (blue dot or colored button) on the handpiece until fluid fills the pumping chamber, and then, while continuing to hold the irrigation button down, press the "HI" button on the controller to power prime the system. Release irrigation button after 3 seconds.  
**Note:** The "PRIME" text and the audio indicator will go off and the flow rate setting LED will light up. The system returns to the flow rate setting in use prior to bag runout. If the "PRIME" text remains lit, and the audio indicator is sounding, the tubing set has not been primed properly. Continue power priming to remove all air from the system.

### Use of Laser Fiber Inserts and Laser Fibers with NEZHAT-DORSEY™ and SmokEvac™ Trumpet Valves

- Select proper size laser fiber insert or fiber insert adapter for the size of the fiber being used and prepare it in compliance with the manufacturer's instructions.
- Remove knurled knob plug at rear of the trumpet valve by turning counterclockwise until unthreaded.
- Insert the laser fiber insert through the threaded fitting in the rear of the body of the trumpet valve and thread it into the body until it is firmly seated. Ensure that the laser fiber insert has the appropriate sealing cap/fiber holder on the end (REF 5102200).
- Place the laser fiber through the fiber holder and advance it through the fiber channel.
- Make sure the fiber holder is firmly attached to the female luer lock fitting at the rear of the laser fiber insert.
- Insert suction/irrigation probe with laser fiber sheath through the laparoscopic cannula. Under appropriate visualization, position probe and advance laser tip to desired location.
- Use appropriate eyewear protection prior to turning on the laser system. Follow all of the recommended manufacturer's instructions and warnings while operating in the laser mode.
- Put knurled knob at rear of the trumpet valve to ensure a closed irrigation system.

### Use with Instruments

The Instrument Insert Probe Tip (REF 5101330) may be used with the trumpet valve to insert 5mm instruments.

- Unthread knurled plug from the distal end of the Trumpet Valve. Replace with appropriate Instrument Insert Adapter (or Universal Adapter) and thread until fully seated.
- Align "Quick Disconnect" Instrument Insert Probe with the "Quick Disconnect" Adapter.
- Slide "Quick Disconnect" Instrument Insert Probe toward the Trumpet Valve body until it fully seats and a double "click" is heard.
- Pass hand instrument through Instrument Insert Adapter (or Universal Adapter) and Instrument Insert Probe. (Retract into cannula when precise suction is required.)

### Disposable Tubing Set Removal

- If using the Dual Spike Adapter (REF 0026110), close the appropriate clamp(s) on the bag spike arms of the disposable tubing set.
- Remove suction tube from suction device
- Remove the pumping chamber from the controller by sliding it upwards.
- Discard the disposable tubing set in accordance with hospital waste disposal practices.  
 **Precaution:** After use, the tubing set, trumpet valve and probe tip are a potential biohazard. Handle and dispose of as required by hospital policy and applicable laws.
- Wipe surface of controller with mild soap and water solution applied with a damp cloth. Store for future use.

## TROUBLESHOOTING TIPS

PROBLEM	POSSIBLE / ACTIONS
Logo LED's do not illuminate when disposable pumping chamber is installed into pumping chamber receptacle.	<ul style="list-style-type: none"> <li>• Connect power cord securely to the IEC power cord receptacle on the back of the controller.</li> <li>• Connect power cord securely to an operable Hospital Grade AC outlet.</li> <li>• Load the pumping chamber of the disposable tubing set properly into the pumping chamber receptacle of the controller (<i>Press firmly downward until the disposable is flush with the top surface of the controller.</i>)</li> <li>• Check fuses and contact DAVOL™ for instructions regarding the correct replacement of interchangeable or detachable parts.</li> <li>• Check the back of the pump chamber to ensure that metallic label is installed.</li> <li>• Install new disposable unit.</li> <li>• Call DAVOL™ Customer Service if problem persists.</li> </ul>
Logo LED's blink after disposable pumping chamber is installed and pump will not start.	<ul style="list-style-type: none"> <li>• System fault detected. Electrical continuity to the disposable pumping chamber may be interrupted.</li> <li>• Install new disposable unit.</li> <li>• Call DAVOL™ Customer Service if problem persists.</li> </ul>
"PRIME" text blinking after system is gravity primed.	<ul style="list-style-type: none"> <li>• Depress the colored irrigation button on trumpet valve. Prime unit to remove all trapped air.</li> <li>• Call DAVOL™ Customer Service if problem persists.</li> </ul>
Logo LED's are illuminated but fluid is not flowing through the system.	<ul style="list-style-type: none"> <li>• Check that the irrigation bag(s) are not empty.</li> <li>• Check that the irrigation bag(s) are spiked completely.</li> <li>• Open the clamp(s) for the appropriate irrigation bags.</li> <li>• Remove any kinks in the flexible tubing. Ensure that the fluid path is open and fluid is flowing toward the trumpet valve handpiece.</li> <li>• Check that the appropriate button (colored button) is depressed on the trumpet valve handpiece.</li> </ul>
Low pressure and flows.	<ul style="list-style-type: none"> <li>• Ensure that the pumping chamber is properly seated into the pumping chamber receptacle of the controller.</li> <li>• Ensure that spike is inserted fully into the irrigation container.</li> <li>• Completely open the clamps on the irrigation bag spike arms.</li> <li>• Remove any kinks in the flexible tubing. Ensure that the fluid path is open and fluid is flowing toward the trumpet valve handpiece.</li> <li>• Install new disposable unit.</li> <li>• Call DAVOL™ Customer Service if problem persists.</li> </ul>
Irrigation pressure does not increase or decrease when pressure settings are selected from the keypad.	<ul style="list-style-type: none"> <li>• Ensure that the pumping chamber is properly seated into the pumping chamber receptacle of the controller.</li> <li>• Completely open the clamps on the irrigation bag spike arms.</li> <li>• Remove any kinks in the flexible tubing. Ensure that the fluid path is open and fluid is flowing toward the trumpet valve handpiece.</li> <li>• Firmly depress irrigation button.</li> <li>• Call DAVOL™ Customer Service if problem persists.</li> </ul>
Motor does not run. Logo and pressure setting buttons are illuminated.	<ul style="list-style-type: none"> <li>• Check that the irrigation bag(s) are not empty.</li> <li>• Remove any kinks in the flexible tubing. Ensure that the fluid path is open and fluid is flowing toward the trumpet valve handpiece.</li> <li>• Ensure that the irrigation button on trumpet valve handpiece is depressed.</li> <li>• Install new disposable unit.</li> </ul>
Motor runs intermittently without cause.	<ul style="list-style-type: none"> <li>• Depress irrigation button. Prime unit to remove all trapped air.</li> <li>• Avoid violent shaking of the irrigation tubing during use.</li> </ul>
Motor runs continuously without cause.	<ul style="list-style-type: none"> <li>• Depress irrigation button. Prime unit to remove all trapped air.</li> <li>• Check for fluid leaking at the trumpet valve handpiece or irrigation tip.</li> <li>• Check for fluid leaking at the pumping chamber.</li> <li>• Call DAVOL™ Customer Service if problem persists.</li> </ul>

	Contents
	Do not use if package is damaged.
	Contains phthalates
	Type B Applied Part

	Biological Risks
	Consult Instructions for Use
	Sterilized using Ethylene Oxide
	Do Not Re-use

Bard, Davol, SmokEvac, Endo-Pool, X-Stream, and Nezhat-Dorsey are trademarks and/or registered trademarks of C. R. Bard, Inc. or an affiliate.

Copyright © 2005, 2008, 2010, 2011, 2014 C. R. Bard, Inc. All Rights Reserved.

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Copyright © 2005, 2008, 2010, 2011, 2014 C. R. Bard, Inc. Tous droits réservés.
Davol Inc.
Filiale de C. R. Bard, Inc.